## TENS Pain Relief Replacement Pads

### Contraindications

- Do not use this TENS device on those who have a cardiac pacemaker, implanted defibrillator, or other implanted metallic or electronic devices as this may cause electric shocks, burns, electrical interference, or death.
- Do not use this TENS device on those whose pain syndromes are undiagnosed.

### Cautions and Warnings

- For best results, it is recommended to replace the electrode pads after 30-50 uses. Use numbers will vary, depending on skin and/or climate conditions as well as pad care, usage, and storage.
- Do not apply stimulation when showering, bathing, or swimming.
- Replace electrode pads when they no longer adhere, become crusted or separated, or weakens stimulation.
- Do not use while operating a motor vehicle.
- If any adverse reaction occurs, discontinue use immediately.
- Check treatment area under pads periodically.

### Application

- Make sure the treatment area is clean of dirt, body lotion and make-up. Dirty or oily skin will prevent electrode pads adhesion.

▶1

- Remove electrode pads from bag. Reseal bag to protect the remaining electrode pads.
- Snap the electrode pads directly onto the control unit, making sure the pads are firmly connected.
- Remove electrode pads from protective film and save the film for electrode pads storage.
- Apply the electrode pads onto the body areas in need of stimulation. Press down firmly and ensure full and firm contact with skin. If necessary, clip any excess body hair with scissors.
- Repeated application may be improved by doing the following:  
Wash the adhesive side of the pad by running it under cold water for one second and then shaking off the excess water. Let air-dry for 1-2 minutes before reapplying to the skin or to the protective film. Do not oversaturate with water as this will reduce adhesiveness.

### Removal and Storage

- Turn off the TENS device before removing the electrode pads from the treatment area.
- Lift a corner of the electrode pad away from the skin and slowly peel it away. Avoid touching the gel as much as possible.
- Place the electrode pad back onto the glossy side of the saved protective film.
- Gently remove the connection wire and button heads from the electrode pads and control unit.
- Remove the electrode pad protective film from the storage bag and seal them tightly on the pads to prevent dry-out.
- Store the TENS device in a cool place out of direct sunlight.

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